Quickstart Guide: Talking Brix™ communicator

ablenet®

www.ablenetinc.com

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www.ablenetinc.com

Programming your Talking Brix™

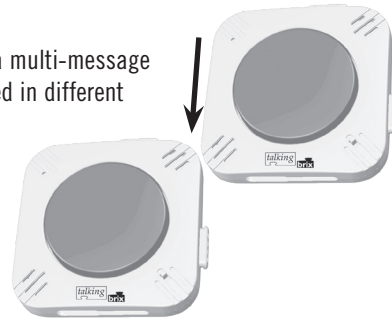
1. Turn the device ON using the power switch on back. (The device's light will flash Red 1x.)
2. Press and Hold the switch top for 5 seconds. (Begin speaking when the device's light blinks Red.) *NOTE: The device will play a portion of any recorded messages.*
3. Release switch top when finished speaking. (The device's light will turn off.)
NOTE: Talking Brix can store up to 10 seconds of audio.
4. Press switch top to PLAY message. (The device's light will flash Red 1x when activated.)

To use picture symbols up to 1.25" (32mm), lift the attached Snap Switch Cap with a fingernail.

Attaching and Mounting

Use the tabs and slots on your Talking Brix to create a multi-message communication device. Individual Brix can be arranged in different configurations to meet the specific needs of the user.

Mount any Talking Brix using the built-in magnet or place Velcro® hook and loop fastener in the recessed area on back.

Powering the Talking Brix

When Talking Brix have a low battery, the light will blink Red 3x after message playback. Attach the Talking Brix to the included wall charger. The light will be solid RED while charging; solid Green when charged.

NOTE: Talking Brix cannot be used while charging.

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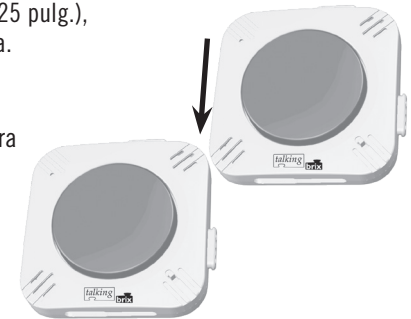
Warranty/Repair/Return Information

AbleNet offers a limited two-year warranty on the Talking Brix communicator product. For full warranty details, visit www.ablenetinc.com

For more information on AbleNet products and services:

AbleNet, Inc., | Minneapolis/St. Paul, MN | 800-322-0956 (US & Canada) | Internet: www.ablenetinc.com

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© 2009 AbleNet, Inc., All Rights Reserved
AbleNet, Inc., | Minneapolis/St. Paul, MN | 800-322-0956 (États-Unis et Canada) | Internet : www.ablenetinc.com
© 2009 AbleNet, Inc., tous droits réservés
AbleNet, Inc., | Minneapolis/St. Paul, MN | 800-322-0956 (USA und Kanada) | Internet : www.ablenetinc.com
© 2009 AbleNet, Inc., Alle Rechte vorbehalten.